



PROGRAMMING YOUR NEW PENDANT

- STEP 1 PRESS AND HOLD DOWN THE GREEN CANCEL KEY ON THE UNIT UNTIL THE UNIT BEEPS TWICE REMOVE FINGER RED LIGHT WILL FLASH AND UNIT WILL ANNOUNCE PROGRAMMING MODE
- STEP 2 PRESS AND HOLD DOWN THE GREEN CANCEL KEY ON THE UNIT UNTIL THE UNIT BEEPS REMOVE FINGER RED WILL CONTINUE TO FLASH
- STEP 3 PRESS RED BUTTON ON NEW PENDANT AND UNIT WILL BEEP AND ANNOUNCE PERSONAL TRIGGER RED LIGHT WILL CONTINUE TO FLASH
- STEP 4 PRESS AND RELEASE GREEN CANCEL BUTTON AND RED LIGHT WILL STOP FLASHING.
DO NOT HOLD DOWN GREEN BUTTON ON THIS STEP
- STEP 5 PRESS RED BUTTON ON NEW PENDANT AND THE UNIT WILL DIAL THROUGH TO OUR MONITORING CENTRE STAFF WILL CALL OUT THROUGH UNIT AND YOU WILL NEED TO ADVISE THEM YOU ARE TESTING NEW PENDANT.

ANY PROBLEMS OR QUERIES PLEASE CONTACT US ON 1850 23 23 24