



## PROGRAMMING YOUR NEW PENDANT

- STEP 1 PRESS AND HOLD DOWN THE GREEN CANCEL KEY ON THE UNIT UNTIL THE UNIT BEEPS TWICE REMOVE FINGER UNIT WILL ANNOUNCE PROGRAMMING MODE.
- STEP 2 PRESS AND HOLD DOWN GREEN CANCEL BUTTON AGAIN UNIT WILL BEEP ONCE REMOVE FINGER UNIT WILL ANNOUNCE REGISTRATION MODE.
- STEP 3 PRESS RED BUTTON ON NEW PENDANT AND UNIT WILL ANNOUNCE PERSONAL TRIGGER REGISTERED.
- STEP 4 PRESS AND RELEASE GREEN CANCEL BUTTON AND RED LIGHT WILL STOP FLASHING.  
**DO NOT HOLD DOWN GREEN BUTTON ON THIS STEP**
- STEP 5 PRESS RED BUTTON ON NEW PENDANT AND THE UNIT WILL DIAL THROUGH TO OUR MONITORING CENTRE STAFF WILL CALL OUT THROUGH UNIT AND YOU WILL NEED TO ADVISE THEM YOU ARE TESTING NEW PENDANT.
- STEP 6 ONCE YOU HAVE SPOKEN TO THE MONITORING CENTRE FROM THE TEST CALL ON PENDANT AND ARE SURE IT IS WORKING PLEASE DISPOSE OF OLD PENDANT IN YOUR WASTE BIN.

ANY PROBLEMS OR QUERIES PLEASE CONTACT US ON 1850 23 23 24