

Quick User Guide for New Pebbell

Contents of Box

Pebbell
Key Ring
Lanyard
Charging Dock
White Cable
Black Charging Block

Getting Started

Connect white cable to back of Charging Dock and then the opposite end to Black Charging Block. Plug into power source and place your Pebbell on the dock with the "O" button facing you. The Pebbell will give a quick vibration to indicate it is on correctly and charging. Charging will take approximately 2 hours (red light on the white base will go off).

Once charged remove Pebbell, press and hold the "O" button for approx. 3 secs and it will vibrate. The Pebbell will sound once and vibrate and then it will give a ring tone, the monitoring operator will then speak through the Pebbell.

Testing the GPS take your Pebbell outside for approx. 5 - 10 mins and then activate the Pebbell as described as above. Ask the Monitoring operator to give you a GPS location, this is only required on the initial test.

Your device is now ready to use.

Get to know your PEBBELL

		SOS button	Call button		
	speaker			LED indicator	on/off
	Charging Contacts	mic		micro USB	

<p>Green LED indicator</p> <p>LED ON (solid) = No SIM card detected</p> <p>Single slow flash every 3 seconds = Device is connecting to GSM network</p>	<p>BLUE LED indicator</p> <p>LED ON (solid) = Device is charging</p> <p>Single rapid flash every 3 seconds = Device has a GPS fix</p> <p>Single slow flash every 3 seconds = Device has no GPS fix</p> <p>LED OFF (when not charging) = GPS is sleeping</p>	<p>Single rapid flash every 3 seconds = Connected to GSM network</p>	<p>Double rapid flash every 3 seconds = Connected to GPRS</p>
---------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------	---------------------------------------------------------------

PEBBELL charging



Note: A wrong charger may damage the device. Use only the original PEBBELL charger. The ideal temperature range for charging is between +10° C and +30° C. The battery life might be shortened if charging the battery below or above these temperatures.

The PEBBELL is equipped with an internal rechargeable battery. The autonomy of the battery may vary depending on the GPS and GSM activity and the use of the device.

- a) Connect the Micro USB side of the cable to the docking station port and connect the other end of the cable to the designated AC power source (USB/AC adaptor).
- b) Place the device on the docking station.
- c) When charging the PEBBELL for the first time, fully charge the battery for at least 3-4 hours until the RED light on the front of the docking station stops flashing and becomes solid.