


To replace the battery

- 1) To access the battery, remove belt/shirt clip accessory from the back of the iVi by turning the clip anti-clockwise until it clicks.

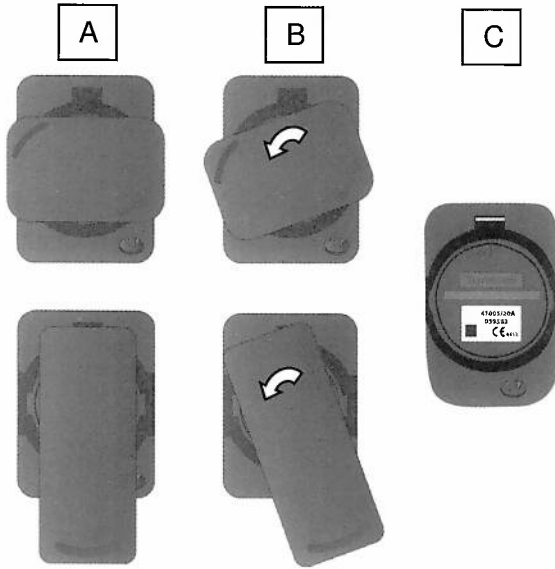

- 2) Remove the battery cover, insert a coin into the slot and turn it anticlockwise.
- 3) Remove the battery and replace it with the new battery (CR2450), remember to observe the correct orientation i.e. positive (+) side facing you. When the battery is reconnected the unit will beep.
- 4) Replace the battery cover and test the iVi by pressing the help button. Inform the operator that you are making a battery test call.
- 5) If the device fails to make a test call then check the orientation of the battery and repeat the test by pressing the help button.

